

Dr. Martin Schwarz Bioscaling

Bioscaling Antibacterial dental Gel which removes dental plaque.



www.DrMartinSchwarz.com/BS/

INGREDIENTS: GLYCERIN, AQUA, SODIUM BICARBONATE, CELLULOSE GUM, POLYSORBATE 80, MENTHOL.
Dr. Martin Schwarz- Bioscaling kit includes: syringes 2x5ml of the gel each.

Bioscaling

Antibacterial dental gel „Bioscaling“ by Dr. Martin Schwarz is a formula designed to remove dental plaque using a method which is completely safe for teeth and tooth enamel. It is intended for everyday dental care. An innovative formula of the Bioscaling product enables removal of plaque consisting of micro-organisms and substances of mineral and organic origin. Due to a suitably selected proportion of microcrystalline sodium bicarbonate and glycerol, Bioscaling gel ensures the excellent aesthetic result and significantly reduces number of bacteria in deposits, remaining in the interdental spaces and the spaces between teeth and gums. The reduction of oral bacterial flora in the mouth results in a number of additional effects that improve the overall well-being and quality of life of a Bioscaling formula user.

The formula of Bioscaling product ensures:

Fresh breath - as bad breath is often caused by micro-organisms found in plaque and in the interdental spaces.
Faster recovery after respiratory diseases – the antibacterial feature of Dr. Martin Schwarz Bioscaling gel accelerates plaque removal and consequently speeds up recovery after prior treatment of respiratory diseases.
Better effect of teeth whitening pastes and gels – elimination of a thin film of bacteria on the tooth enamel makes the products and whitening toothpaste more effective.
More effective with the use of a sonic toothbrush – using the product with a sonic toothbrush improves the effect of the sonic toothbrush

The product is particularly recommended in the case of:

- plaque enlargement
- the occurrence of deposits in the interdental spaces
- bad breath
- low pH level in the mouth
- frequent respiratory diseases
- smoking
- using pastes and whitening products

Before treatment:

- Restore caries lesions and periodontal diseases
- Thoroughly clean, rinse and dry your teeth.

Usage:

1. Before applying Dr. Martin Schwarz Bioscaling product, brush your teeth thoroughly, rinse your mouth with water and dry your teeth.
2. Put a small amount of Bioscaling product in the mouth trays. The amount of gel in the mouth trays should be enough to cover the surfaces of teeth and small enough so that after applying the mouth trays on the teeth, the gel doesn't come in contact with mucous membranes. It is also possible to apply a small amount of the gel directly on a toothbrush for brushing.
3. Mouth trays with Bioscaling product should be placed in your mouth in the right position. If too much Bioscaling product gets in contact with mucous membranes, remove it with a dry toothbrush, a handkerchief or a finger.
4. Wear Bioscaling gel on your teeth for 20-30 minutes. If you feel the burning sensation of the mucous membrane during the process of plaque removal, you should immediately remove the gel from the place of the irritation using a handkerchief.
5. While wearing the mouth trays with Bioscaling product, do not drink, smoke or eat.
6. Avoid swallowing the gel and the liquid after you gargle.
7. After the recommended waiting time, clean your teeth and mouth trays and rinse your mouth to get all of the product out.
8. Clean the mouth trays with a stream of running water using a toothbrush.
9. The best effect is obtained by repeating the process of plaque removal every day (only once a day!) for up to 12 days.
Do not apply Bioscaling gel on decayed or damaged teeth.

Do not use the product if:

- You are under 16
(it is advisable to consult with your dentist if you are under 16 before using the Bioscaling Antibacterial Gel)
- You are pregnant or breastfeeding
- You are taking strong drugs (psychotropic, antiepileptic, cytotoxic)
- You are allergic or hypersensitive to oxidants, glycerol and modified cellulose
- You have mechanical or carious lesions on the tooth surface, exposed teeth roots, temporomandibular disorders

Storage

Product needs to be stored in a cool (25°C and below), dry and dark place - out of sight and reach of children. After each use, wash the mouth trays thoroughly. All the kit elements must be kept clean and dry. Do not use, if any element of the kit is damaged. The kit must be used for its intended purpose and as recommended in the instructions.

Interactions

The use of strong drugs during plaque removal treatment with Bioscaling by Dr. Martin Schwarz requires prior consultation with the treating doctor. If you experience any disturbing symptoms that may be associated with the use of Bioscaling product by Dr. Martin Schwarz, you should stop the treatment and talk to your dentist.

Responsible person:
LDK GmbH
Im Technologiepark 1
Frankfurt Oder, 15236

Info@DrMartinSchwarz.com

DR. MARTIN SCHWARZ

