

Flexi Trays

Mouth trays for independent application of dental gels



www.DrMartinSchwarz.com/FT/

Flexi Trays set by Dr. Martin Schwarz contains: 2 mouth trays, storage box
INGREDIENTS: Ethylene-Vinyl Acetate

Flexi Trays


Flexi Trays by Dr. Martin Schwarz is a set of thermoforming mouth trays designed for an independent application and distribution of various dental specimens on the teeth surface. The shape of a product is easy to be adjusted to the user's teeth through its thermo-fitting features. After adjusting the mouth trays properly, distribute the dental gel into mouth trays and then apply and keep on your teeth for an appropriate period of time. Since the purposes and features of dental gels vary, Flexi Trays should be used in accordance with instructions provided by the gel manufacturer.

The product is particularly recommended for:

- Distribution of whitening gels on the teeth surface
- Application of desensitizing gels
- Use of antibacterial dental gels
- Distribution of other dental specimens

Fitting procedure:

1. Before applying Dr. Martin Schwarz Flexi Trays set with dental gel on teeth, adjust the mouth trays shape.
2. Practice wearing mouth trays. Apply them on your teeth and bit it gently to define their shape.
3. Pick up one Flexi Tray and submerge it into freshly boiled the water for 5-8 seconds.
4. Hold the tab of a tray and place the remaining part of it into the water with a temperature around 80°C. Pay particular attention to not to exceed the recommended 5-8s tray submerging time. If the tray stays longer in the water, it can be melted and destroyed.
5. As the tray soften, make sure to prevent both sides of the tray from touching each other.
6. When the recommended time elapses, take the tray out of the water, wait a second for a tray to cool down and carefully place it on one of your teeth arch. Make sure that the tray covers your teeth and then use your fingers to press the trays against your teeth from both sides. While doing that suck the water and air from the tray out in order to mold the closest fit possible.
7. Remove the tray from your mouth and investigate the result. If the fit is not good enough to repeat the procedure, however, keep in mind that submerging the tray in hot water for too long / too many times may damage it.
8. After receiving a satisfactory fitting result, place the tray in icy cold water to prevent it from changing the desired shape.
9. Trim unnecessary part of the tray e.g. a tab, for comfortable use.
10. Repeat the procedure with the second tray.
11. Apply dental specimen to the Flexi Trays and keep them in the mouth according to instructions given by a dental gel manufacturer

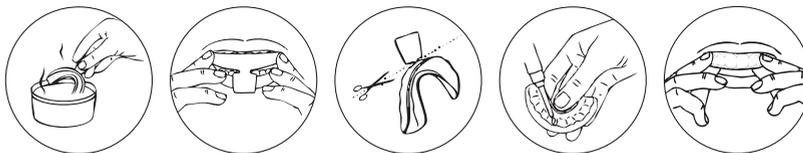

Do not use the product if:

- You are allergic or hypersensitive to products ingredients.
- Application on the product causes pain.
- Size of the product does not fit your jaws.

STORAGE:

Keep away from heat, sparks, and flame. Store in a cool place in the original container and protect from sunlight.

Keep out of sight and reach of children. After each use, wash the mouth trays thoroughly. All the kit elements must be kept clean and dry. Do not use if any element of the kit is damaged.

The kit has to be used for its intended purpose and as recommended in the instructions.

INTERACTIONS:

The product is stable and not reactive under normal conditions of storage and use. Avoid exposure to open flame or exceeding recommended processing conditions.

Responsible person:
LDK GmbH
Im Technologiepark 1
Frankfurt Oder, 15236

Info@DrMartinSchwarz.com

DR. MARTIN SCHWARZ

Dr. M. Schwarz